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Regi-trans®

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Recommendations for use **GB**

Regi-trans®

Regi-trans is a highly precise, transparent bite registration material based on addition-curing vinyl silicones, which is characterised by its particularly simple and time-saving application. It offers the possibility of maintaining visual control of the occlusion during the registration and can also be used to produce transparent moulds for the polymerisation of light-curing composite materials. The application straight out of the cartridge means the material can be applied to the tooth directly and without any bubbles.

INSTRUCTIONS FOR USE

Preparation of the cartridge

To prepare the cartridge for use, insert it in the dispenser before removing and disposing of the cap. Do not replace the cap, as this makes it impossible to rule out cross contamination and thus unintentional partial curing. The green mixing tip is then placed on the cartridge and turned 90° to the right. The material can now be applied immediately without any bubbles and in precisely the correct dosage and mixing ratio. The mixing tip remains on the cartridge as a seal until the next usage.

Registration

The automatically mixed material is applied directly to the occlusal surfaces to be registered. The patient should close his/her mouth immediately after the material is applied, at the latest after 25 seconds (working/application time at 23°C). The material's transparency allows unimpeded visual control of the occlusion during biting. After 1:30 minutes, the material is fully cured and the registration can be removed.

Articulator

The transparent bite registration allows the technician visual control of the occlusion when fixing theplaster model in the articulator. Less than 0.1% shrinkage combined with a final hardness of more than 70 shore A guarantees a perfect occlusion result.

Side effects

No side effects are known to date. Local hypersensitivity reactions to components of Regi-trans cannot be ruled out in individual cases. If swallowed, seek medical advice in case of discomfort or other complaints. Should the product accidentally come into contact with the eyes, rinse thoroughly with water immediately and seek medical advice in case of discomfort or other complaints. Prevent the product from coming into contact with clothing, as it cannot be removed by dry cleaning.

Disinfection

Bite registrations can be disinfected with a 2% glutaraldehyde solution or disinfection solutions, for example. Please follow the manufacturer's instructions.

Storage

Do not store at temperatures above 25°C (78°F)! Do not use after the expiry date! Registrations made with **Regi-trans** can be stored for several months without any shrinkage.

Note

Do not touch the patient's teeth when wearing latex gloves, as latex can affect the setting reaction of the silicones by means of contamination.

Warranty

Bisico, Bielefelder Dentsilsilicone GmbH & Co. KG guarantees that this product is free from material and manufacturing defects. The user is responsible for the application and proper use of the product. If damage is identified in the product within the warranty period, your sole claim and Bisico's sole obligation is to repair or replace the product.

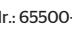
Limitation of liability

Except where prohibited by law, Bisico, Bielefelder Dentsilsilicone GmbH & Co. KG accepts no liability for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or intent.

Technical data	
Consistency	heavy
Mixing time	none
Working time	25 seconds
Intraoral curing time	1:30 minutes
Final hardness	approx. 70 shore A
Linear shrinkage	max. 0.1%

Supply form

[REF] 71900 | 3 x 50 ml cartridges | 18 green mixing tips

<div>Made in Germany! For dental use only! Federal law restricts this device to sale by or on the order of a dentist</div>	<div></div>
<div>Rev. Nr.: 65500-04.2018</div>	<div></div>

Bisico • Bielefelder Dentsilsilicone GmbH & Co. KG • Johanneswerkstraße 3 • D-33611 Bielefeld
Tel.: +49 521 8016800 • Fax: +49 521 8016801 • E-Mail: info@bisico.de

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